


FC-0014

Strapbrackets diagonal

Instructions

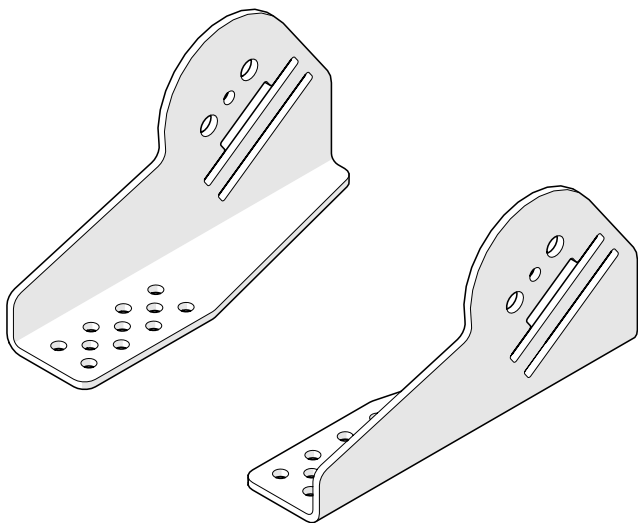


Mounting strapbrackets


Select appropriate fixing for the strapbrackets:

Note:


- Place the diagonal strapbrackets as far back so the straps are behind and below the buttocks of the client.

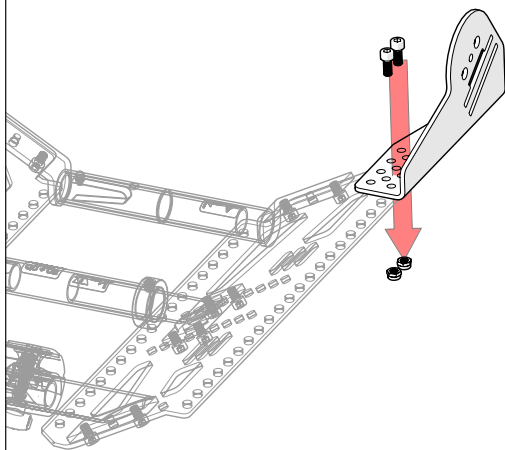

right

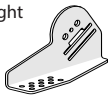

left

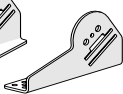

2x Strapbrackets





4x



4x

Cam-buckles

Mount the Cam-buckles to the strapbrackets. Use the holes in the strapbracket.

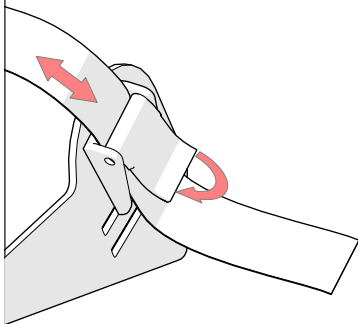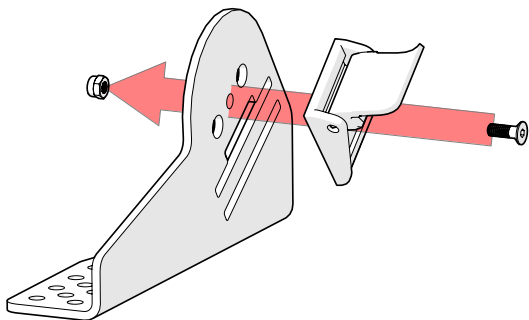

Adjusting Cam-buckle

1. Pull the straps through the Cam-buckles and set the desired length
2. Close the Cam-buckles